

Anti-Measles Virus ELISA (IgG)




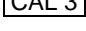
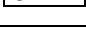












Test instruction

ORDER NO.	ANTIBODIES AGAINST	IG CLASS	SUBSTRATE	FORMAT
EI 2610-9601 G	Measles virus	IgG	Ag-coated microplate wells	96 x 01 (96)

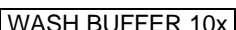Indication: measles

Principles of the test: The ELISA test kit provides a quantitative or semiquantitative in vitro assay for human antibodies of the IgG class against measles virus in serum or plasma. The test kit contains microtiter strips each with 8 break-off reagent wells coated with measles virus antigens. In the first reaction step, diluted patient samples are incubated in the wells. In the case of positive samples, specific IgG antibodies (also IgA and IgM) will bind to the antigens. To detect the bound antibodies, a second incubation is carried out using an enzyme-labelled anti-human IgG (enzyme conjugate) catalysing a colour reaction.

Contents of the test kit:

Component	Colour	Format	Symbol
1. Microplate wells coated with antigens: 12 microplate strips each containing 8 individual break-off wells in a frame, ready for use	---	12 x 8	
2. Calibrator 1 5000 IU/l (IgG, human), ready for use	red coloured in decreasing intensity	1 x 2.0 ml	
3. Calibrator 2 1000 IU/l (IgG, human), ready for use		1 x 2.0 ml	
4. Calibrator 3 250 IU/l (IgG, human), ready for use		1 x 2.0 ml	
5. Calibrator 4 50 IU/l (IgG, human), ready for use		1 x 2.0 ml	
6. Positive control (IgG, human), ready for use	blue	1 x 2.0 ml	
7. Negative control (IgG, human), ready for use	green	1 x 2.0 ml	
8. Enzyme conjugate peroxidase-labelled anti-human IgG (rabbit), ready for use	green	1 x 12 ml	
9. Sample buffer ready for use	light blue	1 x 100 ml	
10. Wash buffer 10x concentrate	colourless	1 x 100 ml	
11. Chromogen/substrate solution TMB/H ₂ O ₂ , ready for use	colourless	1 x 12 ml	
12. Stop solution 0.5 M sulphuric acid, ready for use	colourless	1 x 12 ml	
13. Test instruction	---	1 booklet	
14. Quality control certificate	---	1 protocol	
<div>  Lot  In vitro determination   Storage temperature  Unopened usable until </div>			

Storage and stability: The test kit has to be stored at a temperature between +2°C to +8°C. Do not freeze. Unopened, all test kit components are stable until the indicated expiry date.

Waste disposal: Patient samples, calibrators, controls and incubated microplate strips should be handled as infectious waste. All reagents must be disposed of in accordance with local disposal regulations.



Preparation and stability of the reagents

Note: All reagents must be brought to room temperature (+18°C to +25°C) approx. 30 minutes before use. After first use, the reagents are stable until the indicated expiry date if stored at +2°C to +8°C and protected from contamination, unless stated otherwise below.

- **Coated wells:** Ready for use. Tear open the resealable protective wrapping of the microplate at the recesses above the grip seam. Do not open until the microplate has reached room temperature to prevent the individual strips from moistening. Immediately replace the remaining wells of a partly used microplate in the protective wrapping and tightly seal with the integrated grip seam (Do not remove the desiccant bag).
Once the protective wrapping has been opened for the first time, the wells coated with antigens can be stored in a dry place and at a temperature between +2°C and +8°C for 4 months.
- **Calibrators and controls:** Ready for use. The reagents must be mixed thoroughly before use.
- **Enzyme conjugate:** Ready for use. The enzyme conjugate must be mixed thoroughly before use.
- **Sample buffer:** Ready for use.
- **Wash buffer:** The wash buffer is a 10x concentrate. If crystallization occurs in the concentrated buffer, warm it to 37°C and mix well before diluting. The quantity required should be removed from the bottle using a clean pipette and diluted with deionized or distilled water (1 part reagent plus 9 parts distilled water).
For example: For 1 microplate strip, 5 ml concentrate plus 45 ml water.
The working strength wash buffer is stable for 4 weeks when stored at +2°C to +8°C and handled properly.
- **Chromogen/substrate solution:** Ready for use. Close the bottle immediately after use, as the contents are sensitive to light. The Chromogen/substrate solution must be clear on use. Do not use the solution if it is blue coloured.
- **Stop solution:** Ready for use.

Warning: The controls and calibrators used have been tested negative for HBsAg, anti-HCV, anti-HIV-1 and anti-HIV-2 using enzyme immunoassays and indirect immunofluorescence methods. Nonetheless, all materials should be treated as being a potential infection hazard and should be handled with care. Some of the reagents contain the toxic agent sodium azide. Avoid skin contact.

Preparation and stability of the patient samples

Samples: Human serum or EDTA, heparin or citrate plasma.

Stability: Patient samples to be investigated can generally be stored at +2°C to +8°C for up to 14 days. Diluted samples should be incubated within one working day.

Sample dilution: Patient samples are diluted **1:101** in sample buffer. For example: dilute 10 µl serum in 1.0 ml sample buffer and mix well by vortexing (sample pipettes are not suitable for mixing).

NOTE: Calibrators and controls are prediluted and ready for use, do not dilute them.



Incubation

For **semiquantative analysis** incubate **calibrator 3** along with the positive and negative controls and patient samples. For **quantitative analysis** incubate **calibrators 1 to 4** along with the positive and negative controls and patient samples.

(Partly) manual test performance

Sample incubation: (1st step) Transfer 100 µl of the calibrators, positive and negative controls or diluted patient samples into the individual microplate wells according to the pipetting protocol. Incubate for **30 minutes** at room temperature (+18°C to +25°C).

Washing: Manual: Empty the wells and subsequently wash 3 times using 300 µl of working strength wash buffer for each wash.
Automatic: Wash reagent wells 3 times with 450 µl of working strength wash buffer (program setting: e.g. TECAN Columbus Washer "Overflow Modus").

Leave the wash buffer in each well for 30 to 60 seconds per washing cycle, then empty the wells. After washing (manual and automated tests), thoroughly dispose of all liquid from the microplate by tapping it on absorbent paper with the openings facing downwards to remove all residual wash buffer.

Note: Residual liquid (> 10 µl) in the reagent wells after washing can interfere with the substrate and lead to false low extinction values. Insufficient washing (e.g., less than 3 wash cycles, too small wash buffer volumes, or too short reaction times) can lead to false high extinction values.

Free positions on the microplate strip should be filled with blank wells of the same plate format as that of the parameter to be investigated.

Conjugate incubation: (2nd step) Pipette 100 µl of enzyme conjugate (peroxidase-labelled anti-human IgG) into each of the microplate wells. Incubate for **30 minutes** at room temperature (+18°C to +25°C).

Washing: Empty the wells. Wash as described above.

Substrate incubation: (3rd step) Pipette 100 µl of chromogen/substrate solution into each of the microplate wells. Incubate for **15 minutes** at room temperature (+18°C to +25°C) (protect from direct sunlight).

Stopping the reaction: Pipette 100 µl of stop solution into each of the microplate wells in the same order and at the same speed as the chromogen/substrate solution was introduced.

Measurement: **Photometric measurement** of the colour intensity should be made at a wavelength of 450 nm and a reference wavelength between 620 nm and 650 nm **within 30 minutes of adding the stop solution**. Prior to measuring, slightly shake the microplate to ensure a homogeneous distribution of the solution.

Test performance using fully automated analysis devices

Sample dilution and test performance are carried out fully automatically using the analysis device. The incubation conditions programmed in the respective software authorised by EUROIMMUN may deviate slightly from the specifications given in the ELISA test instruction. However, these conditions were validated in respect of the combination of the EUROIMMUN Analyzer I or the DSX from Dynex and this EUROIMMUN ELISA. Validation documents are available on inquiry.

Automated test performance using other fully automated, open system analysis devices is possible, however, the combination should be validated by the user.



Pipetting protocol

	1	2	3	4	5	6	7	8	9	10	11	12
A	C 3	P 6	P 14	P 22			C 1	P 3	P 11	P 19		
B	pos.	P 7	P 15	P 23			C 2	P 4	P 12	P 20		
C	neg.	P 8	P 16	P 24			C 3	P 5	P 13	P 21		
D	P 1	P 9	P 17				C 4	P 6	P 14	P 22		
E	P 2	P 10	P 18				pos.	P 7	P 15	P 23		
F	P 3	P 11	P 19				neg.	P 8	P 16	P 24		
G	P 4	P 12	P 20				P 1	P 9	P 17			
H	P 5	P 13	P 21				P 2	P 10	P 18			

The pipetting protocol for microtiter strips 1-4 is an example for the **semiquantitative analysis** of 24 patient samples (P 1 to P 24).

The pipetting protocol for microtiter strips 7-10 is an example for the **quantitative analysis** of 24 patient samples (P 1 to P 24).

The calibrators (C 1 to C 4), the positive (pos.) and negative (neg.) controls, and the patient samples have each been incubated in one well. The reliability of the ELISA test can be improved by duplicate determinations for each sample.

The reagent wells are break-off format. Therefore, the number of tests performed can be matched to the number of samples, minimizing reagent wastage.

Both positive and negative controls serve as internal controls for the reliability of the test procedure. They should be assayed with each test run.

Calculation of results

Semiquantitative: Results can be evaluated semiquantitatively by calculating a ratio of the extinction value of the control or patient sample over the extinction value of calibrator 3. Calculate the ratio according to the following formula:


$$\frac{\text{Extinction of the control or patient sample}}{\text{Extinction of calibrator 3}} = \text{Ratio}$$

EUROIMMUN recommends interpreting results as follows:

Ratio <0.8:	negative
Ratio ≥0.8 to <1.1:	borderline
Ratio ≥1.1:	positive

In cases of borderline test results, an additional patient sample should be taken 7 days later and re-tested in parallel with the first patient sample. The results of both samples allow proper evaluation of titer changes.

Quantitative: The standard curve from which the concentration of antibodies in the patient samples can be taken is obtained by point-to-point plotting of the extinction values measured for the 4 calibrators against the corresponding units (linear/linear). Use "point-to-point" plotting for calculation of the standard curve by computer. The following plot is an example of a typical calibration curve. Please do not use this curve for the determination of antibody concentrations in patient samples.

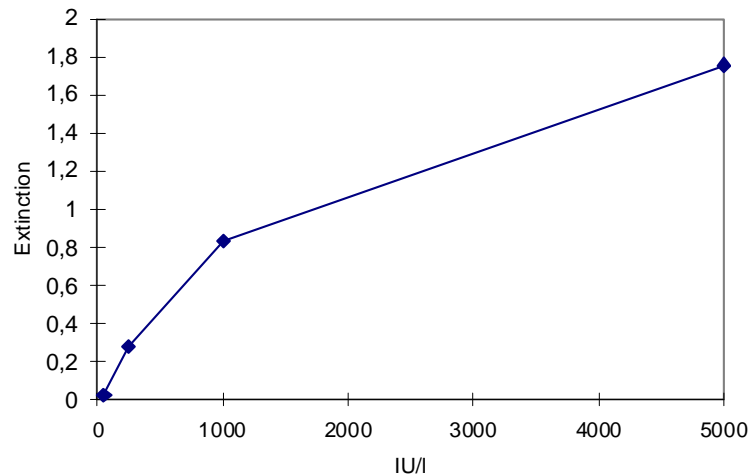

If the extinction of a serum sample lies above the value of calibrator 1 (5000 IU/l), the result should be given as ">5000 IU/l". It is recommended that the sample be re-tested at a dilution of 1:400. The result in IU/l read from the calibration curve for this sample must then be multiplied by a factor of 4.

The upper limit of the reference range of non-infected persons (**cut-off value**) recommended by EUROIMMUN is **250 International Units (IU/l)**. EUROIMMUN recommends interpreting results as follows:

<200 IU/l:	negative
≥200 to <275 IU/l:	borderline
≥275 IU/l:	positive

Evaluation information: For duplicate determinations the mean of the two values should be taken. If the two values deviate substantially from one another the sample should be retested.

For the interpretation of borderline results an investigation using further tests (e.g. avidity determination of antibody class IgG) can be helpful. Diagnosis can be secured by the determination of the titer change in two serum samples taken at an interval of at least 7 days and analysed in parallel.

For diagnosis, the clinical symptoms of the patient should always be taken into account along with the serological results.

Test characteristics

Calibration: The controls of the Anti-Measles Virus ELISA (IgG) were calibrated using the 3rd international standard serum NIBSC 97/648 (anti-measles and anti-polio virus serum, National Institute for Biological Standards and Control, Hertfordshire, England; approved as international reference preparation by the WHO Expert Committee on Biological Standardization). The NIBSC 97/648 serum contains 3 International Units (IU) per ampoule by definition and was resuspended in a concentration of 3 IU/ml.

For every group of tests performed, the extinction values of the calibrators and the international units determined for the positive and negative controls must lie within the limits stated for the relevant test kit lot. A quality control certificate containing these reference values is included. If the values specified for the controls are not achieved, the test results may be inaccurate and the test should be repeated.



The activity of the enzyme used is temperature-dependent and the extinction values may vary if a thermostat is not used. The higher the room temperature during substrate incubation, the greater will be the extinction values. Corresponding variations apply also to the incubation times. However, the calibrators are subject to the same influences, with the result that such variations will be largely compensated in the calculation of the result.

Antigen: The antigen source is provided by inactivated cell lysates of Vero cells infected with the "Edmonston" strain of measles viruses.

Linearity: The linearity of the anti-measles viruses ELISA (IgG) was determined by assaying 4 serial dilutions of different patient samples. The coefficient of determination R^2 for all sera was > 0.95 . The Anti-Measles Virus ELISA (IgG) is linear at least in the tested concentration range (52 IU/l - 4865 IU/l).

Detection limit: The lower detection limit is defined as the mean value of an analyte-free sample plus three times the standard deviation and is the smallest detectable antibody titer. The lower detection limit of the Anti-Measles Virus ELISA (IgG) is 8 IU/l.

Cross reactivity: The quality of the antigen used ensures a high specificity of the ELISA. Sera from patients with infections caused by various agents were investigated with the EUROIMMUN Anti-measles virus ELISA (IgG)

Antibodies against	n	Anti-Measles virus ELISA (IgG)
Adenovirus	8	0%
CMV	6	0%
EBV-CA	11	0%
HSV-1	3	0%
Influenza virus type A	5	0%
Influenza virus type B	11	0%
Mumps virus	4	0%
Mycoplasma pneumoniae	4	0%
Parainfluenza virus types 1-4	11	0%
RSV	9	0%
Rubella virus	6	0%
Toxoplasma	3	0%
VZV	5	0%

Interference: Haemolytic, lipaemic and icteric samples showed no influence on the result up to a concentration of 10 mg/ml for hemoglobin, 20 mg/ml for triglycerides and 0.4 mg/ml for bilirubin in this ELISA.

Reproducibility: The reproducibility of the test was investigated by determining the intra- and inter-assay coefficients of variation using 3 sera. The intra-assay CVs are based on 20 determinations and the inter-assay CVs on 4 determinations performed in 6 different test runs.

<i>Intra-assay variation, n = 20</i>		
Serum	Mean value (IU/l)	CV (%)
1	830	8.0
2	3410	6.6
3	3725	5.6

<i>Inter-assay variation, n = 4 x 6</i>		
Serum	Mean value (IU/l)	CV (%)
1	796	11.6
2	3635	5.0
3	3946	6.8



Specificity and sensitivity: 112 clinically characterized patient samples (interlaboratory test samples from INSTAND, Germany, Labquality, Finland and NEQAS, UK) were examined with the EUROIMMUN Anti-Measles Virus ELISA (IgG). The test showed a specificity and a sensitivity of 100% each.

n = 112		INSTAND / Labquality / NEQAS (IgG)		
		positive	borderline	negative
EUROIMMUN Anti-Measles Virus ELISA (IgG)	positive	89	1	0
	borderline	0	0	0
	negative	0	0	22

Reference range: The levels of anti-measles virus antibodies (IgG) were analyzed with this EUROIMMUN ELISA in a panel of 500 healthy blood donors. With a cut-off of 250 IU/l, 94% of the blood donors were anti-measles virus positive (IgG), which reflects the known percentage of infections in adults.

Clinical significance

The measles virus (MV) is the most instantly recognizable member of Morbilliviruses, a group of viruses belonging to the Paramyxoviridae family [1]. No animal reservoir is known. The measles virus causes an acute feverish illness which occurs mainly in childhood and is very infectious [2, 3]. In 1999, measles still caused worldwide 873,000 deaths per year [1, 4, 5]. Today they are less frequent because of vaccination, especially in the western hemisphere [6, 7]. However, measles epidemics are still observed in some countries [2, 3, 4, 5, 6, 8, 9]. Individuals acutely infected with the virus exhibit a wide range of clinical symptoms ranging from a characteristic mild self-limiting infection to death [1, 2, 8, 10].

MV infections are characterised by an incubation period of about 10 days, flue-like symptoms with fever, malaise, catarrh of the upper respiratory tract, cough, congestion and conjunctivitis. Soon afterwards the measles rash, a typical exanthema, appears first near the ears, then on the forehead, in the face and over the rest of the body [1, 5, 8, 11].

Complications arising from MV infections include secondary bacterial pneumonia, otitis media (approx. 1%), encephalitis (approx. 1%), myocarditis, miscarriage and a condition called subacute sclerosing panencephalitis (SSPE) [5, 12, 13]. Persistent MV infection of the otic capsule is an aetiological factor in otosclerosis [9, 14]. Anti-measles IgG for the serological diagnosis of otosclerotic hearing loss has a high specificity and sensitivity [15]. SSPE is a progressive, generally fatal brain disorder caused by chronic measles virus infection. It occurs about 7 to 10 years after the infection and generally kills within 3 years from the onset of the symptoms. The patients suffer from behavioural changes, cognitive deterioration, vision problems and eventually advanced neurological symptoms, such as severe spasms, and finally severe physical and mental impairment that leads to death [13]. Males are more commonly affected than females. The risk of SSPE from measles was underestimated according to older data [12]. Actual papers put it at closer to 6.5 to 11 cases of SSPE per 100,000 measles infections; that means 7 to 13 times higher than the earlier estimates [1, 9, 12].

Women with acute measles infection during pregnancy and a negative result for measles-specific antibodies were observed e.g. in Japan, India, Thailand, Kenya and Brazil: 3 of 4 pregnancies ended in preterm delivery, spontaneous abortion or stillbirth; 2 of 4 neonates were found to have congenital measles with a positive result for IgM antibodies [5].

Antibodies against MV can be found in the serum of almost all patients during and after a measles infection. IgM antibodies develop soon after the onset of symptoms and can be measured using ELISA or indirect immunofluorescence tests (IIFT) [16, 20, 21, 22]. 50% of patients have IgM antibodies within three days, more than 90% within 10 days after occurrence of the rash [15, 17]. The Anti-Measles Virus IgM ELISA is more rapid and sensitive for the serological diagnosis of measles infections than other tests [15, 18, 19].



MV infections often cause an increase in heterologic antibodies. The statistically evaluated detection rate for antibodies is significantly higher for ELISA and IIFT in comparison with e.g. neutralisation tests [16, 20]. IgG and IgM antibodies against MV are reliable markers to confirm suspected measles infections.

Measles myelitis or encephalitis can be verified by detecting antibodies against measles in the cerebrospinal fluid (CSF) [23, 24, 25, 26]. These specific antibodies are synthesised in the brain [24]. The CSF-serum quotient (LSQ) allows to differentiate between a blood-derived and a pathological, brain-derived specific antibody fraction in CSF, taking into account individual changes in the blood/CSF barrier function [24, 25, 26, 27]. Therefore it is necessary to confirm the presence of antibodies against MV using ELISA both in CSF and in the serum. During measles myelitis or encephalitis an intrathecal synthesis of antibodies against MV in CSF takes place. Due to the fact that specific antibodies can pass the blood-cerebrospinal fluid barrier by diffusion from serum to CSF it is necessary to determine the relative CSF/serum quotient (CSQrel., synonym: antibody specificity index) [24, 25, 26]. The quotient is calculated from the amount of specific anti-measles virus IgG antibodies in total CSF IgG in proportion to the amount of specific IgG antibodies in total serum IgG. During conversion the CSF/serum quotient of the pathogen-specific IgG-antibody concentrations CSQpath.-spec. (IgG) is put into relation to the CSF/serum quotient of the total IgG concentrations CSQtotal (IgG) [27]. A relative CSQ result above 1.5 indicates the production of specific antibodies in the central nervous system (CNS) and the involvement of the CNS in the disease [25, 26].

With respect to the severe complications known from measles infections, the Robert Koch Institute in Germany recommends vaccinating small children, with a first shot between the age of 11 to 14 months and a second between 15 and 23 months [2, 4, 10, 11]. Neutralisation activity and persistence of antibodies are induced in response to the immunisation [6, 15].

Life-long immunity is generally developed. However, antibody levels are 8 to 10 times lower in post-vaccination sera than in convalescent sera [6, 19, 28]. A passive immunisation with specific immunoglobulin concentrates is usually given to immunosuppressed seronegative individuals, such as tumour patients and recipients of transplants, as well as to seronegative pregnant women after exposure to the virus.

The European Regional Office of the WHO aims at eliminating measles from the region in the following years by area-wide vaccination campaigns [4, 9, 10]. This is expected to limit the number of apparent infections and especially of severe courses of the disease. For the diagnosis of the remaining cases of measles infection and of infections acquired outside Europe as well as for the clarification of atypical courses of the disease in partly immunised patients the antibody determination in serum and CSF will be of growing importance [3, 5, 8, 9].



Literature references

1. Rima BK, Duprex WP. **Morbilliviruses and human disease.** J Pathol 208 (2006) 199-214.
2. Hogg GG, Darlington RJ, Hogg KG, Lester R. **Measles immunity and immunisation status in Australian children 1 to 4 years of age.** J Paediatr Child Health 42 (2006) 165-169.
3. Karimi A, Arjomandi A, Alborzi A, Rasouli M, Kadivar MR, Obood B, Pourabbas B. **Prevalence of measles antibody in children of different ages in Shiraz, Islamic Republic of Iran.** East Mediterr Health J 10 (2004) 468-473.
4. Cutts FT, Steinglass R. **Should measles be eradicated.** Br Med J 316 (1998) 765-767.
5. de Barros EN, Silva EM. **Epidemiologic surveillance of measles and rubella in Campinas (SP), Brazil: the reliability of the data.** [Article in Portuguese] Rev Panam Salud Publica 19 (2006) 172-178.
6. Gay N, Ramsay M, Cohen B, Hesketh L, Morgan-Capner P, Brown D, Miller E. **The epidemiology of measles in England and Wales since the 1994 vaccination campaign.** Commun Dis Rep CDR Rev 7 (1997) 17-21.
7. de Melker H, Pebody RG, Edmunds WJ, Levy-Bruhl D, Valle M, Rota MC, Salmaso S, van den Hof S, Berbers G, Saliou P, Spaendonck MCV, Crovari P, Davidkin I, Gabutti G, Hesketh L, Morgan-Capner P, Plesner AM, Raux M, Tische A, Miller E. **The seroepidemiology of measles in Western Europe.** Epidemiol Infect 126 (2001) 249-259.
8. Ceylan A, Ertem M, Korukluoglu G, Acemoglu H, Kara IH, Erten PG, Arslan C, Ay ME. **An epidemic caused by measles virus type D6 in Turkey.** Turk J Pediatr 47(2005) 309-315.
9. Ota MO, Moss WJ, Griffin DE. **Emerging diseases: measles.** J Neurovirol 11 (2005) 447-454.
10. Shann F. **A little bit of measles does you good.** Br Med J 219 (1999) 4-5.
11. CDC **Measles outbreaks still occur among school-age children and travelers.** MMWR 46 (1997) 242-245.
12. Smith TC. **Measles vaccine doesn't cause SSPE.** Aetiology (2006).
13. Yokoyama T, Sakurai M, Aota Y, Wakabayashi Y, Ohyashiki K. **An adult case of acute disseminated encephalomyelitis accompanied with measles infection.** Intern Med 44 (2005) 1204-1205.
14. Singh MP, Ratho RK, Panda N, Mishra B. **Otosclerosis - do we have a viral aetiology?** Nepal Med Coll J 7 (2005) 129-130.
15. Tische A, Gerike E, Strauss J, Smelhausova M, Mrazova M. **Immunoglobulin specific and conventional methods in the serodiagnosis of measles.** [Article in German] Z Gesamte Hyg 35 (1989) 367-369.
16. Roodbari F, Roustai MH, Mostafaie A, Soleimanjdahi H, Foroshani RS, Sabahi F. **Development of an enzyme-linked immunosorbent assay for immunoglobulin M antibodies against measles virus.** Clin Diagn Lab Immunol 10 (2003) 439-442.
17. Rossier E, Miller H, McCulloch B, Sullivan L, Ward K. **Comparison of immunofluorescence and enzyme immunoassay for detection of measles-specific immunoglobulin M antibody.** J Clin Microbiol 29 (1991) 1069-1071.
18. Pedersen IR, Antonsdottir A, Evald T, Mordhorst CH. **Detection of measles IgM antibodies by enzyme-linked immunosorbent assay (ELISA).** Acta Pathol Microbiol Immunol Scand [B] 90 (1982) 153-160.
19. Glikmann G, Petersen I, Mordhorst CH. **Prevalence of IgG-antibodies to mumps and measles virus in non-vaccinated children.** Dan Med Bull 35 (1988) 185-187.



20. Bouche FB, Brons NH, Houard S, Schneider F, Muller CP. **Evaluation of hemagglutinin protein-specific immunoglobulin M for diagnosis of measles by an enzyme-linked immunosorbent assay based on recombinant protein produced in a high-efficiency mammalian expression system.** J Clin Microbiol 36 (1998) 3509-3513.
21. Stöcker* W, Fauer* H, Krause* C, Barth E, Martinez A (*EUROIMMUN AG). **Verfahren zur Optimierung der automatischen Fluoreszenzerkennung in der Immundiagnostik.** Deutsche Patentanmeldung (Offenlegungsschrift) DE 10 2006 027 516.0 und WO2007140952 (2006).
22. EUROIMMUN AG. **Aktuelle Themen der Autoimmundiagnostik und der Infektions-Serologie.** Wissenschaftliches Fortbildungsseminar mit Vorträgen von Prof. Dr. G. Wick, Prof. Dr. N. Sepp, Prof. Dr. F. Deisenhammer, Dr. med. W. Stöcker, Prof. Dr. Gerold Stanek, Prof. DDr. R. Würzner, A. Krapf, Dr. med. Dipl. oec. med. J. Brunner, Innsbruck (2007).
23. Dennin RH, Herb E. **Immunological diagnosis in viral infections of the central nervous system: course of antibody titres against homo- and heterologous viruses.** Med Microbiol Immunol 178 (1989) 255-268.
24. Reiber HO, Lange P. **Quantification of Virus-Specific Antibodies in Cerebrospinal Fluid and Serum: Sensitive and Specific Detection of Antibody Synthesis in Brain.** Clin Chem 37 (1991) 1153-1160.
25. Reiber H, Peter JB. **Cerebrospinal fluid analysis: disease-related data patterns and evaluation programs.** J Neurol Sci 184 (2001) 101-122.
26. Reiber H, Lange P. **Virus-spezifische Antikörper in Liquor und Serum. ELISA-Analytik und Auswertung mittels Antikörper-Index und Quotientendiagramm.** Lab Med 15 (1991) 204-207.
27. Reiber H, Ungefehr S, Jacobi C. **The intrathecal, polyspecific and oligoclonal immune response in multiple sclerosis.** Multiple Sclerosis 4 (1998) 111-117.
28. Dine MS, Hutchins SS, Thomas A, Williams I, Bellini WJ, Redd SC. **Persistence of vaccine-induced antibody to measles 26-33 years after vaccination.** J Infect Dis 189 (2004) 123-130.



